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****ENGLISH - Important Safety Warning - Electrical Stimulation****

****WARNING: **** Using an electrical stimulation device carries potential risks. Please read and understand the following instructions carefully before use.

*** **Contraindications: **** Do not use this device if you have a pacemaker, are pregnant, suffer from heart conditions, epilepsy, blood clotting disorders, or have skin lesions on the application area. Consult a doctor if in doubt. *** **Electrode Placement: **** Place the electrodes only on clean, dry, and intact skin. Never place them on the head, neck, chest, or directly over the heart. *** **Intensity: **** Always start with the lowest intensity and gradually increase to a comfortable level. A tingling sensation is normal, but pain is unacceptable. Stop immediately if you experience pain. *** **Duration and Frequency: **** Do not exceed the duration and frequency of use recommended in the user manual. Excessive use may lead to skin irritation or muscle pain. *** **Children: **** Keep this device out of reach of children. *** **Maintenance: **** Clean the electrodes regularly after each use with a damp cloth. Never immerse the device in water. *** **Adverse Effects: **** In case of skin irritation, redness, burns, or any other adverse reaction, discontinue use immediately and consult a doctor.

This document is generated automatically in response to the European product safety regulations and in accordance with the GDPR.

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